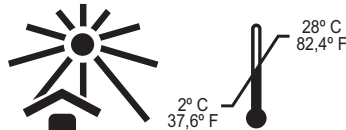
FORMOCRESOL

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Reg. ANVISA: 10298550076.

COMPOSITION:

Formaldehyde, ortho-Cresol, Glycerin; 96° Ethyl Alcohol.

INDICATION:

For conservative treatment of primary dentition and in some specific cases of permanent teeth;

TECHNICAL INFORMATION:

Its formulation attends the concepts adopted by the current researches being, therefore, more diluted than the standard composition of Buckley, but it has the same or superior effectiveness, and a larger biological compatibility;

Its function is to fasten the alive pulps, maintaining them inert and facilitating the conservation in deciduous tooth until a close time of the physiologic fall (pulpotomy);

It has a potent antibacterial action by its components action, which justifies its use in long curative in endodontics treatment.

USE INSTRUCTIONS:

1. Anesthetize and isolate the tooth with a rubber dam;
2. Remove all decayed tissue;
3. Remove cavity pulp roof;
4. Remove coronal pulp (pulpotomy);
5. Rinse cavity pulp using detergent and humectant irrigation solution (a.e. Tergental or Tergipol);
6. Apply **FORMOCRESOL** on pulp stump using a cotton pellet, and wait for 5 to 7 minutes until the hemostasis;
7. Remove the cotton pellet and apply a layer of Zinc Oxide and Eugenol and other with Zinc Phosphate Cement. After restore with amalgam or steel crown.

SPECIAL CARE:

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

FORMOCRESOL should not be used in patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.
"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

FOR USE IN DENTISTRY ONLY