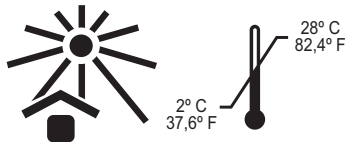
TRICRESOLFORMALINA

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TRICRESOLFORMALINA

Material for Root Canal Disinfection

biodinámica®

CONTENTS:

1x10mL flask **TRICRESOLFORMALINA**.

Reg. ANVISA: 10298550075.

COMPOSITION:

Formaldehyde; Ortho-Cresol; 96º Ethyl Alcohol and Deionized Water.

INDICATION:

As a long term curative in cases of pulpal necrosis.

TECHNICAL INFORMATION:

Antiseptic, disinfectant, that combines the property of Formaldehyde with ortho-Cresol;

USE INSTRUCTIONS:

1. After root canal is instrumented, soak a cotton ball in **TRICRESOLFORMALINA** and remove excess;
2. Place in the pulp cavity;
3. Close hermetically with appropriate cement;
4. After 24 - 48hs, remove the curative and fill root canal.

SPECIAL CARES:

TRICRESOLFORMALINA contain toxic substances in its formulation.

Ingestion of small quantities may cause nausea, vomit, convulsion and abdominal colic.

Dispose the product in accordance with local regulations, ensuring its complete distortion, preventing its reuse and environment damages.

CONTRAINDICATION:

TRICRESOLFORMALINA should not be used in patients is known to have sensibility to any of the ingredients listed.

WARNINGS:

Do not use material after expiry date.

The manufacturer is not liable for damages caused by improper use or use not provided at these instructions.
"KEEP MATERIAL OUT OF THE REACH OF CHILDREN".

FOR USE IN DENTISTRY ONLY