



## HEMOSTANK Hemostatic For Topical Use

**PRESENTATION:**

1x5ml or 10 ml flask **HEMOSTANK SOLUTION**

**COMPOSITION:**

HEMOSTANK Solution : Aluminium Chloride 25%, Aluminium hidroxiquinoleina 0,1% and deionized water. All the components in enough quantity for 100%

HEMOSTANK Gel: Aluminium Chloride 25%, Aluminium hidroxiquinoleina 0,1%, deionized water and thickner. All the components in enough quantity for 100%

**INDICATION:**

HEMOSTANK is used in cases where is needed a precision in the border's gum molding, during cavities prepare and it is used together with the retractor cord. It may be also used to control a local bleeding , when necessary.

**TECHNICAL INFORMATION:**

HEMOSTANK is a hemostatic agent that acts for vasoconstriction and proteic coagulation (collagen protein precipitation), for the Chloride Aluminium action presents in it's formutalion . It's possesses astringent action and do not possess epinephrine in it's composition.

**INSTRUCTIONS FOR USE:**

I – As a helper in molding and in cavity preparations:

Humidify the retractor cord with the HEMOSTANK solution and then proceed in accordance to the indicated technique.

II – For exudation and gingival hemorrhage:

Place one or two drops of the solution in the gum, using a little cotton ball. Wait 1 or 2 minutes and rinse carefully all the area where the product was applied

**SPECIAL CARES:**

Use only the necessary quantity of drops and after getting the desired result, rinse abundantly the local to remove the product.

The product is irritating for the eyes and for the mucosa

Do not use it in a retractor cord already impregnated

Do not swallow

**CONTRA- INDICATION:**

HEMOSTANK should not be used in patients are known to have sensibility to any of the ingredients listed